Invacare® P550/2 Actio 2™

en	Rollator User Manual	1
de	Rollator Gebrauchsanweisung	5
es	Rollator Manual del usuario	10
fr	Rollator Manuel d'utilisation	14
it	Rollator Manuale d'uso	18
nl	Rollator Gebruiksaanwijzing	22
pt	Andarilho Manual de utilização	26

General

About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING
Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION
Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT
Indicates a hazardous situation that could result in damage to property if it is not avoided.

en

Tips and Recommendations
Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Intended use

The rollator is intended to support a person during walking and may also serve as a resting seat.

The maximum user weight is stated in the chapter "Technical Data" and on the product label.

Indications

The product is intended for support and improvement of a limited walking ability.

The user's remaining walking and cognitive ability needs to be sufficient for safe usage.

Contraindications

The product is not intended for use by persons with severe equilibrium disorders or functional perceptual disorders.

Permissible operating conditions and locations

For indoor use only (in private or public areas).

Not to be used under water in e.g. pools or open-air bath, neither in hot areas as e.g. saunas.

Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.


This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

Yes, you can.®

Scope of delivery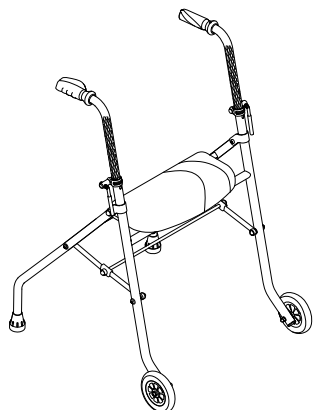

- Rollator, partly assembled
- User manual

Warranty

We supply a two year warranty from the date of delivery. Deviations may occur due to local agreements. Damage caused by normal wear to e.g. wheels, handles and brakes or damage caused by abnormal use (e.g. exceeded max. user weight) is not covered by the warranty. Maintenance and operating instructions must be followed for the warranty to be valid.

Safety

General safety information

WARNING!

Risk of serious injury

- Incorrect use can lead to hazardous situations
- Do not use the product on stairs or on escalators
 - Do not stand on the product
 - Be careful when walking on sloping/slippery ground or uneven surfaces
 - Avoid walking against obstacles, both forwards and backwards
 - Do not let children play with the product

WARNING!

Risk of falling

- Incorrect use can lead to hazardous situations
- Do not use the product to transport a person.
 - Do not maneuver the product with your feet while sitting on it
 - Make sure that no clothes, such as long skirts, can get stuck in the wheels

WARNING!

Risk of injury

- Unbalanced or heavy load can overturn the product
- Do not hang any loads, such as bags, to the frame or push handle
 - Only carry loads in the basket, tray or on the loading shelf, if applicable
 - Do not exceed the max. load indicated for the tray, basket or loading shelf (for maximum load, see chapter “Technical data”)

CAUTION!

Risk of first degree burns

- Parts of the product may heat up, when exposed to high temperatures (above 41° C)
- Be careful if the product is exposed to high temperatures


IMPORTANT

Risk of corrosion

- Do not store the product in wet or humid environments

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.


Labels and symbols on the product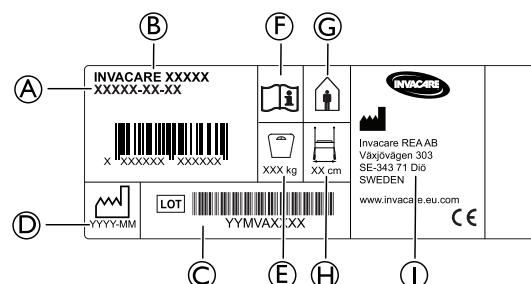

The label contains all important technical information regarding your walking aid.

- | | |
|-----------------------|-----------------------------|
| Ⓐ Article number | Ⓕ Refer to user manual |
| Ⓑ Product name | Ⓖ Indoor walking aid |
| Ⓒ Lot number | Ⓗ Max. width of walking aid |
| Ⓓ Date of manufacture | Ⓘ Manufacturer's name. |
| Ⓔ Max. user weight | |

Placement of serial label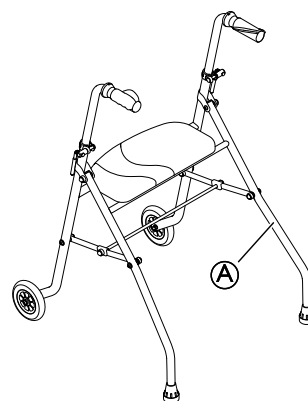

The serial label is placed on the rear tube Ⓐ.

Usage

Before use

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.

Operating position


WARNING!

Risk of falling


- Make sure that the folding mechanism is in a locked position before using the rollator.

1. Remove the plastic caps in the side tubes, if any.
- 2.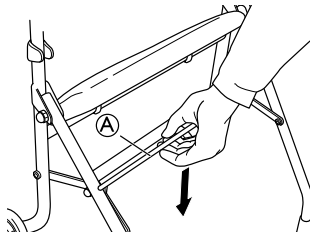


Open the rollator and press the folding mechanism downwards for operating position.

3.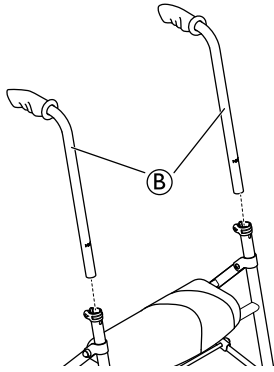


Assemble the push handles (B).

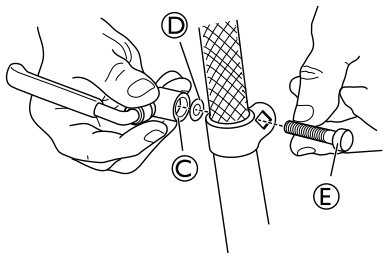4. Assemble the lever (C), the washer (D) and the bolt (E) with the locking lever on the outer side of the tube.


 Ensure that the bolt fits correctly into the square shaped hole on the attachment.

5.

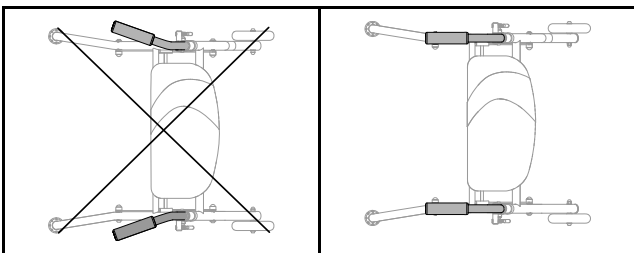 **WARNING!**
Risk of falling
 – The handlebars must not be angled outwards as this reduces the lateral stability of the rollator.

6.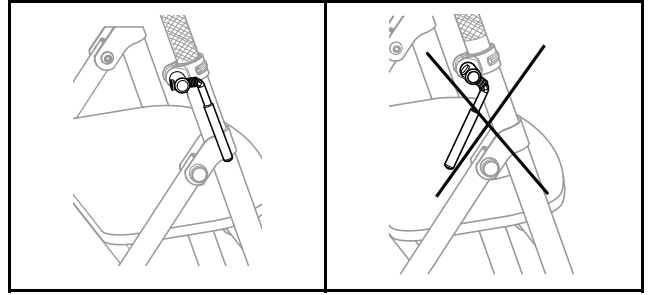

Tighten the locking lever.

 The locking levers must be pointing downwards during use.

7. Repeat the procedure on the opposite side.

Unfolding and folding the rollator


Unfolding the rollator

WARNING!


Risk of falling

– Make sure that the folding mechanism is in a locked position before using the rollator.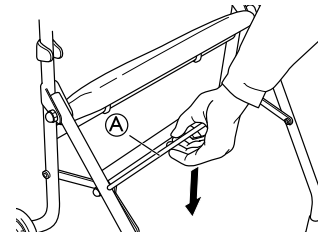


1. Unfold the rollator.
2. Push the folding mechanism (A) downwards.

Folding the rollator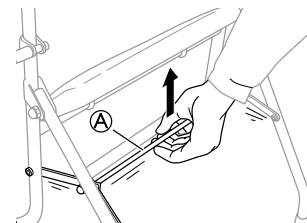

1. Pull the folding mechanism (A) upwards.
2. Fold the rollator.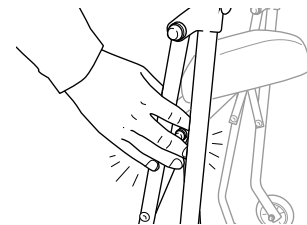

WARNING!


Risk of pinching

– Be aware of the pinching risk when folding the rollator.

Adjusting the handle height/angle

The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.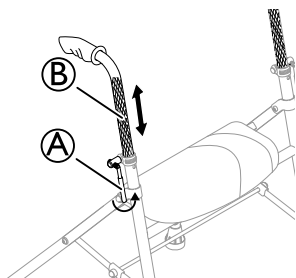


1. Stand behind the rollator with flat shoulders.
2. Loosen the lever (A).
3. Adjust the handle (B) to suitable height.

WARNING!
User may fall

– Make sure that the push handles are not adjusted more than maximum. See the marking on the push handle tube.

4. Re-tighten the lever.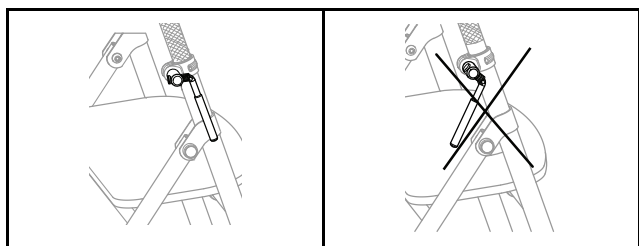


The locking levers must be pointing downwards during use.

5. Repeat the procedure on the other side.

Using the seat


The seat cushion can be used as a resting seat.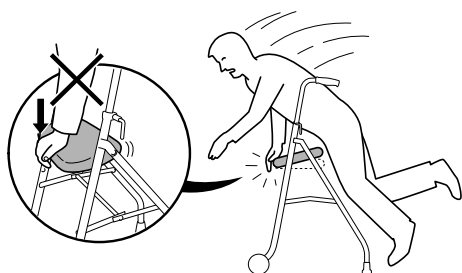

WARNING!
Risk of falling

– Do not lean on or sit on the front edge of the seat as the seat may open.

Mounting the basket

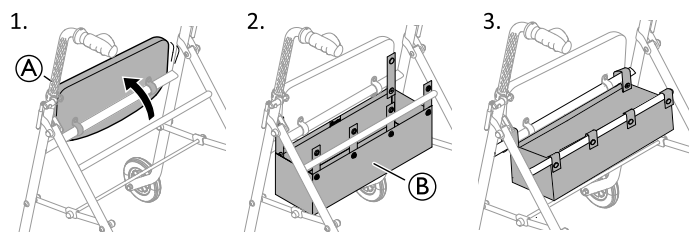1. Fold up the seat cushion (A).
2. Place the basket (B) between the cross tubes.

3. Fold the attachment straps around the cross tubes and close the buttons.
4. Fold the seat back down again.

See section “Dimensions and weight” for information about max load capacity.

Maintenance and care

Maintenance

Maintenance should be done at least every six months.

To ensure good performance and safety you should regularly check that the following is carried out:

- Ensure that screws and adjustments are properly tightened if applicable.
- Never use a defective rollator. Do not attempt repairs yourself!
- If hand grips are loose do not use the rollator.
- Periodically inspect the wheels, for wear and damage when applicable.
- Make sure that the wheels are free of hair, lint and other debris when applicable.
- Ensure that the rubber tips and/or plastic glide tips are not ripped, worn or missing when applicable. Replace the tips and/or plastic glide tips if necessary before using the rollator.

Cleaning

The product can be cleaned and disinfected using commercially available detergents.

IMPORTANT

- Cleaning must be done at least once a year.
- Do not use an abrasive detergent cleaner.
- Never use acids, alkalis or solvents.

- The product can be washed by hand.
- The washing and drying temperature may not be higher than 60°C.
- High-pressure cleaner or ordinary jet of water may be used. However, do not aim the jet towards seat or labels.
- Clean with damp cloth and mild detergents.
- Clean the wheels with a brush (no wire brush!) when applicable.
- After washing, wipe the product dry.

Disinfection

Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

After Use

Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way.

The individual parts can be separated and handled according to their material.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Re-use

This product is suitable for re-use after the following has been performed:

- Inspection
- Cleaning

For details, see chapter *Maintenance and care*, page 4.

WARNING!
Risk of injury


- Always inspect the product according to the service instructions before transferring the product to a different user.
- If any damage is found do not use the product.

The user manual belongs to the product. Make sure that the user manual will be handed over with the product.

Technical Data

Dimensions and weight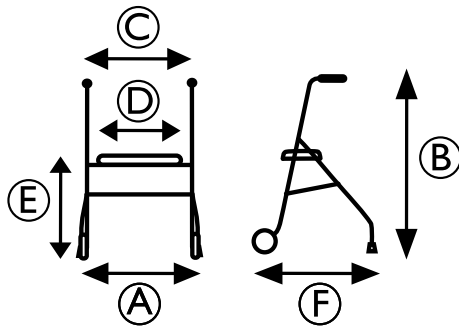

	P550/2 ACTIO 2
Total width Ⓐ	530 mm
Total height Ⓑ	740 – 1000 mm
Total width between push handles Ⓒ	410 mm
Total seat width Ⓓ	380 mm
Total seat height Ⓔ	550 mm
Total length Ⓕ	630 mm
Maximum user weight	130 kg *
Total weight without accessories	5 kg
Maximum load basket	2,5 kg
Diameter front wheels	110 mm

* The maximum user weight is the weight limitation of the product and includes the contents of the basket and the weight of the user.

Material

Frame	Polyester powder coated steel
Handle bars	Chrome plated steel
Handgrips	PVC
Wheels	Polypropylene with treads of TPE
Screws	Steel, rust protected

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Invacare Rea AB